


„Lolly One“

New Generation Covid19 Antigen Selftest

Spring SARS-CoV-2 Antigen Saliva B


Pleasant testing for self-application. Excellent scores in the test results. Minimally invasive. It recognizes all currently known variants. Easy and safe handling. Test result with only one action. Do not transfer or dab liquids. Suitable for all Age groups, including for children. Observe the instructions for use.

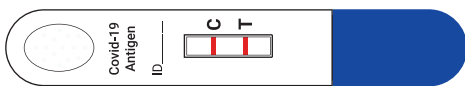

„All in one“ self test



Swiss Made Technology

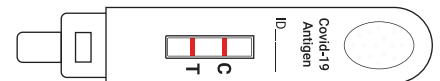

How the Lolly One works:

Cough vigorously three times, remove the cap, take the tip of the Lolly One into the Mouth, gently stroke the pad 10 times in a circular motion the inside of your cheek and tongue. Keep the Lolly One in your mouth for about two minutes with a lot of saliva. Cover the rectangular hole with your lips and blow vigorously several times into the test cassette until the liquid rises. Place the cap on the tester and place it on a clean surface. Wait until you can read the test result on the C and T line.



+41 (0)41 552 50 60



info@springhealthcare.org



Turmstrasse 30
6312 Steinhausen
Switzerland